

# unica anterior



powered by:  
**STYLE ITALIANO™**

the one  
solution  
for anterior  
restorations

matrix  
systems



**polydentia**  
swiss manufacture 



1 Restoration of both, interproximal and cervical, margins at once

2 Contoured profile for an anatomic restoration of the interproximal margins

3 Placement wings for ease of positioning and matrix adaptation

4 Easy management of the cervical area: predictable restoration and/or gingival retraction

# unica anterior



*"In today's conservative dentistry, the choice of products and the techniques we apply everyday must be predictable and cost-effective, incorporating clear-cut advantages to our daily procedures. Unica is the solution to everyday anterior restorations, whether it be class III, IV, V, shape modifications, and/or composite veneers in direct stratification technique. One effective matrix for everyday anterior aesthetic and functional results."*

Dr. Jordi Manauta, DDS  
Head of Styleitaliano Idea Factory  
Sestri Levante, Italy

Unica is the simple and ideal matrix for anterior restorations such as class III, IV, V, direct stratification composite veneers, and shape modifications.

Thanks to its contoured shape, Unica adapts correctly to the different morphologies of anterior teeth and makes it possible to restore proximal and cervical margins at once, even in presence of rubber-dam or gingival retraction cords, thus reducing chair time significantly. The placement wings allow fast and efficient matrix positioning.

Furthermore, Unica anterior once positioned allows to easily visualise the final shape of the restoration, even before starting the procedure.



## Composite veneers

for more information



### kit



#### Unica Introkit anterior - REF 6900

- 35 pcs Unica anterior matrices
- 40 pcs myWedge XS
- 40 pcs myWedge S
- 40 pcs myWedge M
- 1 pc myCustom Resin, 3 ml syringe
- 7 pcs needle tips gauge 18 form myCustom Resin
- 1 pc myQuickmatrix Forceps

### refill



Unica anterior matrices  
REF 6912 12 pcs  
REF 6950 50 pcs



REF 5703  
myQuickmatrix Forceps



REF 7101  
myWedge Extrasmall (XS) 100 pcs



REF 7102  
myWedge Small (S) 100 pcs



REF 7103  
myWedge Medium (M) 100 pcs



REF 6811  
myCustom Resin, 3ml Syringe & 10 pcs. needle tips gauge 18



# Class III restoration



In case of a Class III cavity where the proximal margin is missing, Unica is very useful as it allows the clinician to visualise the final shape even before proceeding with the restoration.

Dr. Giuseppe Chiodera  
Brescia, Italy



for more information



# Class IV restoration



For this Class IV restoration, Unica helps to restore the cervical and interproximal profiles anatomically and in a very fast way.

Dr. Dan Lazar  
Oradea, Romania



for more information



Unica anterior matrices

myQuickmatrix Forceps




myCustom resin


myWedge



**Yes**

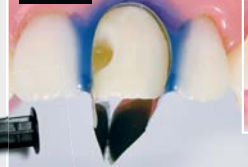

**No**

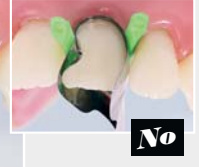

Fix Unica anterior with myCustom Resin when the interdental wedge would risk deforming and/or moving the matrix.



# Class V restoration



In case of Class V restorations, the contoured shape of the matrix helps the dentist identify and visualise the cervical profile before proceeding with the treatment.

The anatomically shaped cervical collar allows also for a comfortable access to the operator field.

Dr. Giuseppe Chiodera  
Brescia, Italy



for more information

# Shape modification




The unique design of Unica makes this matrix an essential and very efficient tool since it enables us to work both in the interproximal and cervical areas at the same time.

Dr. Carlos Fernández Villares  
Madrid, Spain



for more information



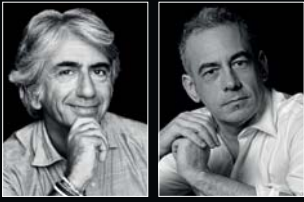

# Styleitaliano

founded by  
Dr. Walter Devoto and  
Prof. Angelo Putignano

*Style Italiano* is a group of passionate clinicians and researchers, who are in constant development of materials and techniques for aesthetic and conservative dentistry. Their philosophy is based on developing solutions to make excellence in aesthetic and functional dentistry attainable by all.

For Polydentia, the collaboration with Style Italiano is more than just a source of inspiration. It is an opportunity to constantly renew its commitment to put excellence in Swiss manufacturing at the service of dentistry worldwide.



Visit “Unica  
Community”  
to share  
*Unica anterior*  
clinical cases

Follow us  
**#thinkunica**



Polydentia SA  
via Cantonale 47  
6805 Mezzovico-Vira  
Switzerland

phone: +41 91 946 29 48  
fax: +41 91 946 32 03

info@polydentia.ch  
www.polydentia.ch

