

SDI

WAVE

FLUORIDE RELEASING
FLOWABLE COMPOSITE




REGULAR,
MEDIUM
AND HIGH
VISCOSITY

A FLOWABLE FOR ALL YOUR CLINICAL SITUATIONS

Wave, Wave MV and Wave HV are ideal versatile, fluoride releasing, radiopaque, light cured flowable composites designed for anterior and posterior restorations. Wave is an extremely low viscosity flowable, Wave MV is a medium viscosity and Wave HV is a high viscosity flowable composite.

A CHOICE OF 3 VISCOSITIES

The wide number of viscosities available in the Wave range ensures the clinician has a choice for all different clinical conditions.

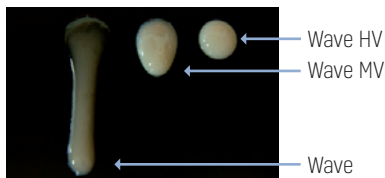

Flow at 90° angle for 2 minutes

| SDI Flowable | Compare to |
|--------------|-------------------------------|
| Wave | UltraSeal XT Plus* |
| | Premise Flowable* |
| Wave MV | Filtek Supreme Plus Flowable* |
| | Heliomolar Flow* |
| | Tetric Flow* |
| Wave HV | Revolution Formula 2* |
| | Flow-It! ALC* |
| | Point 4 Flowable* |
| | Filtek Flow* |
| | AeliteFlo LV* |
| | Esthet-X Flow* |
| | AeliteFlo* |


14
SHADES



40%
LONGER TIPS

FLUORIDE RELEASE

Fluoride's cariostatic effect enhances remineralization and inhibits enamel demineralization. In an aqueous environment, the fluoride ions in Wave, Wave MV and Wave HV diffuse from the resin into the surrounding tooth. This ionic movement is caused by oral fluid passing in and out of the resin and tooth, acting as a carrier for the fluoride ions.

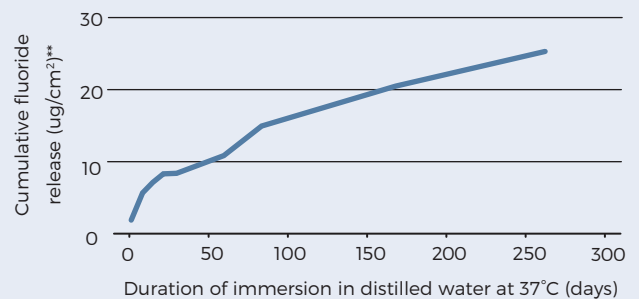


COMPRESSIVE STRENGTH

COMPRESSIVE STRENGTH

Wave's, Wave MV's and Wave HV's high compressive strength values expand the suitability for multiple applications.


MPa @ 24 HOURS**

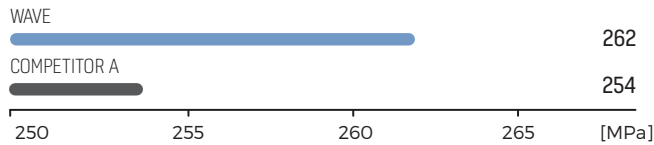

FILLER LOADING


Wave, Wave MV and Wave HV all have excellent wear profiles due to their high filler loading level:

% BY WEIGHT**

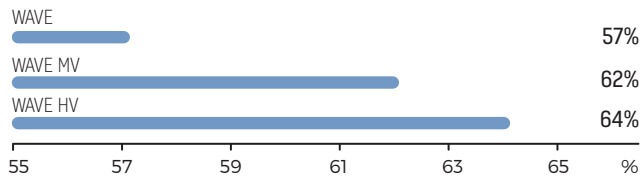

TIP LENGTH COMPARISON

Wave's complete tip length is 12.2mm, 40% longer than the alternative of 8.7mm. Wave's longer tip length allows more direct placement of the material into difficult to reach areas.

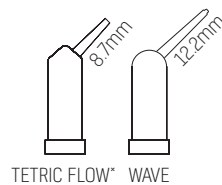

NANOFILLED

Wave, Wave MV and Wave HV contain specially treated nano-fillers to maximize polishability, wear resistance and strength. Ease of extrusion is improved with the addition of these specially treated nano-fillers.

RADIOPACITY

Wave, Wave MV and Wave HV are all visible on x-rays. This allows easy identification of secondary caries at the interface.

ADVANTAGES

High compressive strength

Nano technology

Three viscosities

Natural fluorescence


No after cure shade change

SPECIALTY SHADES

Wave's 14 shades are ideal for all anterior and posterior restorations: A1, A2, A3, A3.5, B1, B2, B3, C2, C3, D3, OA2, incisal, gingival and bleach. Gingival shade is ideal for restoring cervical lesions in elderly patients. For teeth lighter than B1, Bleach shade is available. A semi-translucent shade, Incisal, is perfect as a final layer for composite restorations.

NATURAL FLUORESCENCE

Wave, Wave MV and Wave HV contain a fluorescent agent that ensures perfectly natural looking teeth under ultraviolet lights. No one will know!

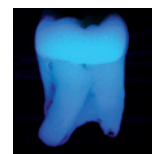

Wave on natural teeth under UV light

INSTRUCTIONS

- 1** Clean and isolate tooth
- 2** Remove excess water. Preferably keep moist, but a dry surface is ok
- 3** Apply Go! to saturate all internal surfaces and leave in place for 20 seconds. Or, apply other bonding agent according to manufacturer's instructions
- 4** Blow firmly with dry, oil free air for 5 seconds. Leave surface glossy
- 5** Light cure for 10 seconds

- 6** Directly inject Wave in increments of 2mm or less in:
 - 6.1** Class V restorations or
 - 6.2** Pit and fissure sealants or
- 6.3** Tunnel preparations or
- 6.4** Minimal Class I, Class II, Class III and Class IV restorations, or other indications as required
- 7** Cure Wave for 20 seconds in increments of 2mm
- 8** Polish and finish

ORDER DETAILS

SYRINGES

Wave Syringe Refills

1 x 1g Syringe
5 Single Use Disposable Tips

| | WAVE | WAVE MV | WAVE HV |
|---------------|---------|---------|---------|
| A1 | 7510102 | 8310102 | 8210102 |
| A2 | 7510203 | 8310203 | 8210203 |
| A3 | 7510304 | 8310304 | 8210304 |
| A3.5 | 7510405 | 8310405 | 8210405 |
| B1 | 7511101 | 8311101 | 8211101 |
| B2 | 7511211 | | |
| B3 | 7511314 | | |
| C2 | 7512223 | | |
| C3 | 7512324 | | |
| D3 | 7513334 | | |
| OA2 | 7510204 | | |
| Incisal | 7514142 | | |
| Gingival (A5) | 7514143 | | |
| Bleach | 7514147 | 8214147 | |

COMPLETS

Wave Complet Refills

20 x 0.25g Complets

| | WAVE | WAVE MV |
|------|---------|---------|
| A2 | 7520002 | 8320002 |
| A3 | 7520003 | 8320003 |
| A3.5 | 7520004 | |

TIPS

Disposable Tips for Wave Syringes

| | |
|---------------------------------------------|---------|
| 20 x Single Use Disposable Tips (20 Gauge) | 7500001 |
| 200 x Single Use Disposable Tips (20 Gauge) | 8100062 |

Disposable Tips for Wave MV & HV Syringes

| | |
|----------------------------------------------|---------|
| • 20 x Single Use Disposable Tips (18 Gauge) | 8100065 |
|----------------------------------------------|---------|

KITS

Wave Syringe Introductory Kit

4 x 1g Syringes A2, B3, C3, OA2
20 Single Use Disposable Tips

7500000

Wave MV Syringe Introductory Kit

4 x 1g Syringes A2, B2, C2, OA2
20 Single Use Disposable Tips

8300000

Wave Syringe Bulk Kits

10 x 1g Syringes
40 Single Use Disposable Tips

| | |
|----|---------|
| A1 | 7500011 |
| A2 | 7500012 |
| A3 | 7500013 |

* Not the registered trademarks of SDI Limited.

** Source-Published and SDI test data.



YOUR SMILE. OUR VISION.



MADE IN AUSTRALIA
by SDI Limited
Bayswater, Victoria 3153
Australia
www.sdi.com.au

AUSTRALIA 1800 337 003
AUSTRIA 00800 0225 5734
BRAZIL 0800 770 1735
FRANCE 00800 0225 5734
GERMANY 0800 100 5759

ITALY 00800 0225 5734
NEW ZEALAND +64 21 400 323
SPAIN 00800 0225 5734
UNITED KINGDOM 00800 0225 5734
USA & CANADA 1 800 228 5166